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Additional Information for MammoDuo Model

This product comprises two monitor units mounted on a dedicated stand. Some methods used to handle this product differ from those described in the Instructions for Use. Before using the product, read this document and the Instructions for Use carefully to ensure that the product is used correctly.


PRECAUTIONS

WARNING

Carry or place the unit according to the correct specified methods.

Dropping the unit may result in injury or equipment damage. Be sure to observe the following.

- When unpacking and / or carrying the monitor, ensure at least two people are utilized.
- Grasp and firmly hold the bottom and handle of the monitor.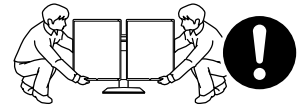

When removing the monitor unit from the stand, be sure to utilize two people.

Removing only one of the monitors from the stand unit will make the product extremely unstable. Working on the product in an unstable state may cause the product to topple, which may result in injury or equipment damage.

Package Contents

The following items are supplied together with the product. Check that all of them are contained in the package. If any of these are missing, contact your dealer or local EIZO representative.

RX560

- Monitor
(Monitor unit x 2, stand)
- Power cord x 2
- Digital signal cable: PP300 x 2
DisplayPort - DisplayPort
- Digital signal cable: PP100 x 1
DisplayPort - DisplayPort
- Digital signal cable: DD300DL x 2
DVI - DVI (dual link)
- USB cable: UU300 x 2
- EIZO LCD Utility Disk (CD-ROM)
- Instructions for Use
- Additional Information for MammoDuo Model (this document)


GX560

- Monitor
(Monitor unit x 2, stand)
- Power cord x 2
- Digital signal cable: PP300 x 4
DisplayPort - DisplayPort
- Digital signal cable: PP100 x 1
DisplayPort - DisplayPort
- USB cable: UU300 x 4
- EIZO LCD Utility Disk (CD-ROM)
- Instructions for Use
- Additional Information for MammoDuo Model (this document)


Controls and Functions

For information about the part names and functions of the monitor unit, refer to "Controls and Functions" in the Instructions for Use.

Rear view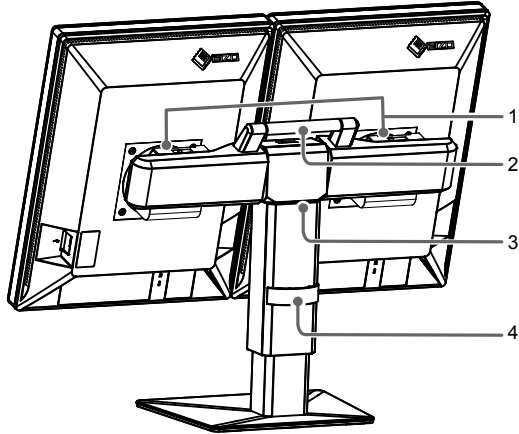

Top view

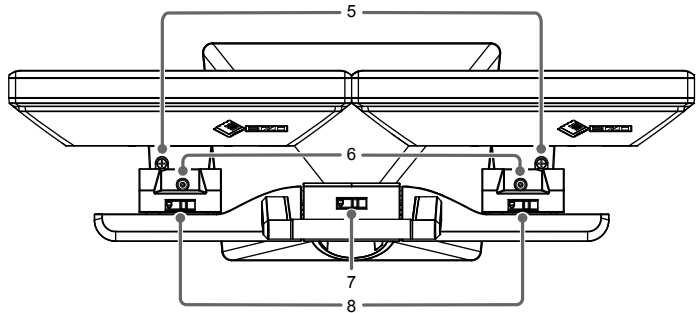1. Mount unit	Connects the monitor unit to the stand.
2. Handle	This handle is used for transportation.
3. Stand	The stand enables you to: <ul style="list-style-type: none"> • Adjust the position and horizontal angle of each monitor. • Adjust the vertical / horizontal angle and height of the entire monitor
4. Cable holder	Holds the monitor cables.
5. Rotation screw	By rotating this screw, each monitor screen can be turned clockwise or counterclockwise to adjust the size of the space between the monitors.
6. Swivel lock screw	Tightening this screw locks the horizontal angle. <p>Note</p> <ul style="list-style-type: none"> • The screw is not locked before shipment.
7. Tilt lock	Locks the vertical angle of the monitor to 0°. <p>🔒 : Locked, 🔓 : Released</p> <p>Note</p> <ul style="list-style-type: none"> • The angle is locked to 0° before shipment. To change the monitor tilt angle, release the lock. • When transporting the product, set the vertical angle of the monitor to 0° and apply the tilt lock. This helps make transportation of the product easier.
8. Mount lock	Locks the mount unit to the stand unit. <p>To remove the mount unit from the stand unit, release the mount lock.</p> <p>🔒 : Locked, 🔓 : Released</p>

Installation / Connection

● Before Installing the Product

Carefully read "PRECAUTIONS" in this document as well as "PRECAUTIONS" in the Instructions for Use.

Installation Requirements

When installing the product, ensure that there is adequate space around the sides, back and top of the monitor.

Attention

- Position the monitor so that there is no light to interfere with the screen.

● Connecting Cables

For information about the compatible resolutions of the monitor, refer to "Compatible Resolutions" in the Instructions for Use.

1. Connect signal cables.


Check the shapes of the connectors, and connect the cables. After connecting the DVI cable, tighten the fasteners to secure the connector.

Attention


- Be sure to use the cables attached to the product.
- The DisplayPort connectors of the monitor consist of input and output connectors. When connecting the monitor to a PC, connect the cable to the input connector.
- To set up a daisy-chain connection, it is necessary to set the DisplayPort version to "1.2". For details on how to perform setup, refer to the Installation Manual (on the CD-ROM).
- Visit the EIZO website for information about monitors and graphic boards that can be used for the daisy-chain connection: <http://www.eizoglobal.com>
- Before using a daisy-chain connection, remove the cap of the output connector on the monitor that outputs signals.

Connection examples:

Connecting each monitor to a PC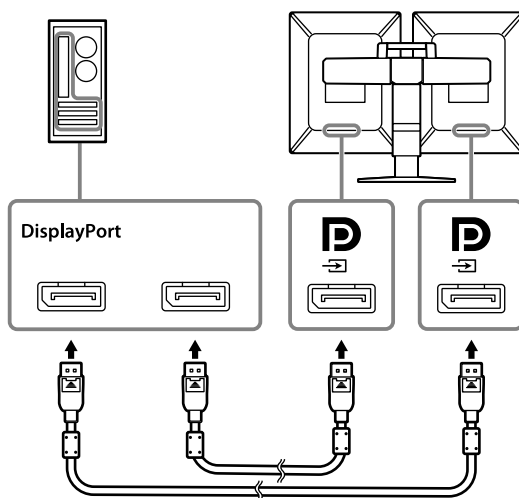

Setting up a daisy-chain connection

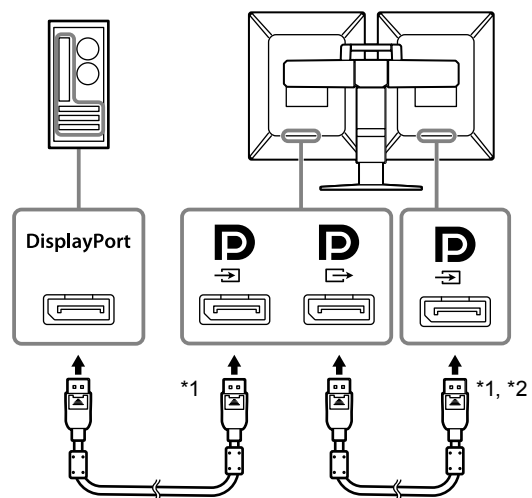
*1 For the GX560, connect to .


*2 Use attached "PP100".

2. Plug the power cord into a power outlet and the power connector on the monitor.

3. Connect the USB cable when using RadiCS / RadiCS LE.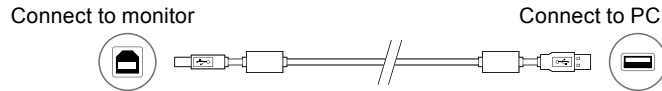

Attention

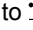- In order to perform monitor quality control for the GX560, connect a PC with RadiCS / RadiCS LE installed to . Refer to "2-2. Connecting Cables" in the Instructions for Use on how to connect cables.
-

4. Touch  on the front of the monitor to turn on the power to the monitor.
The power switch indicator of the monitor lights up green.
 5. Turn on the PC.
The screen image appears.
-

Note

- If the monitor does not operate correctly, refer to "No-Picture Problem" in the Instructions for Use.
-

● Adjusting the Posture of Each Monitor


1. Adjust the swivel and horizontal positions.

Holding the top and bottom of the monitor unit with both hands, adjust the angle and position of each monitor individually.

The swivel angle can be locked by turning the swivel lock screw clockwise to tighten it.

Note

- The swivel lock screw is not locked before shipment.
-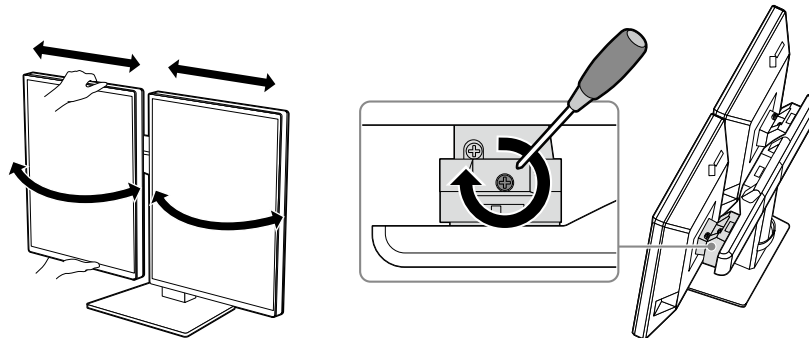


2. Adjust the rotation.

The space between monitors can be eliminated by turning the rotation screw. Adjust as necessary.

Attention

- Do not perform rotation by moving the monitors by hand.
-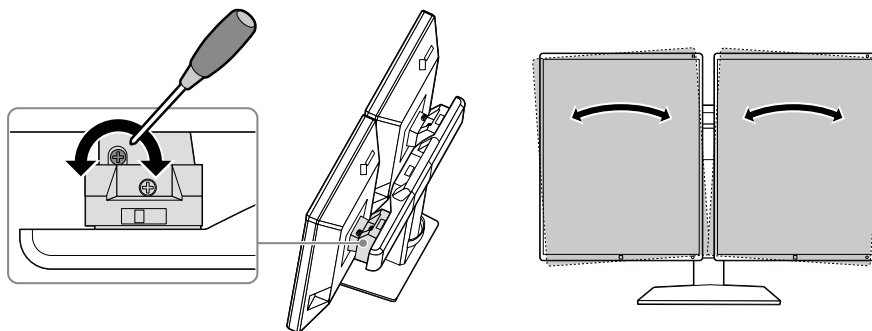


Note

- Use a (#2) phillips head screwdriver for the swivel lock screw and rotation screw.
-


● Adjusting the Screen Height and Angle

Holding both monitors together as in the following figure, adjust the screen height and vertical angle.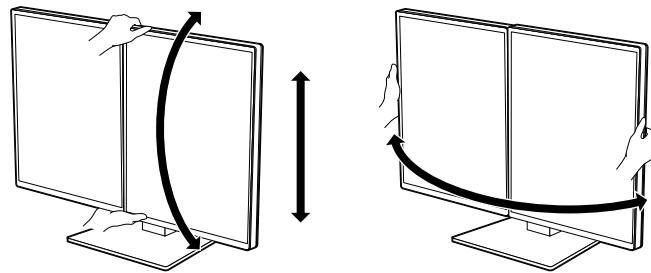

Attention

- The tilt lock is locked before shipment. Adjust the vertical angle of the screens after releasing the lock.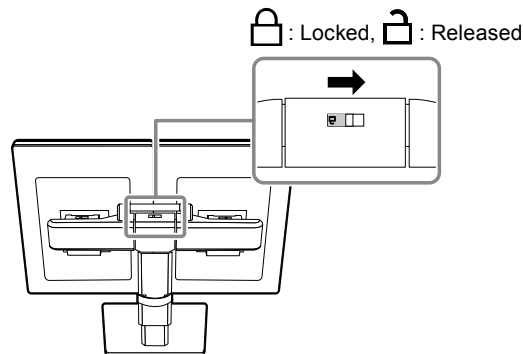

- After adjusting the height and vertical angle, pass the cables through the cable holder.

Reference


● Removing the Monitor

In a case such as when you mount the optional RadiLight on the monitor, remove the monitor unit from the stand.

Attention

- Removing only one of the monitors from the stand will make the product extremely unstable. Working on the product in an unstable state may cause the product to topple, which may result in injury or equipment damage. Be sure to utilize two people.

Remove the monitor unit from the stand after releasing the mount lock.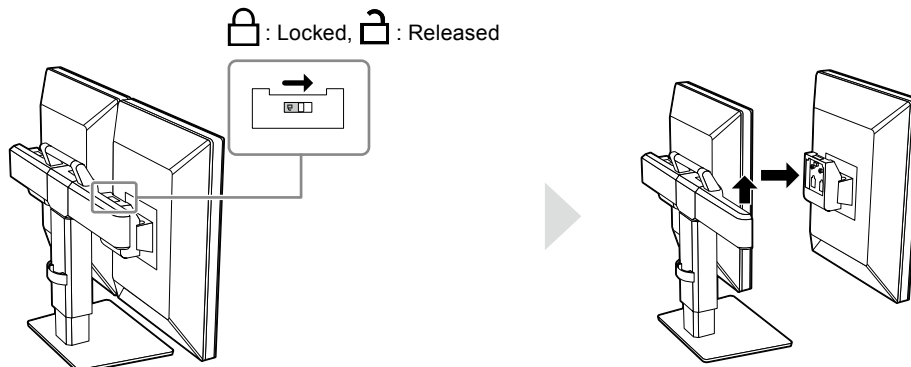


Note

- Refer to the RadiLight Instructions for Use for information about the mounting procedure.

● Mounting the Monitor

Mount the monitor unit on the stand.

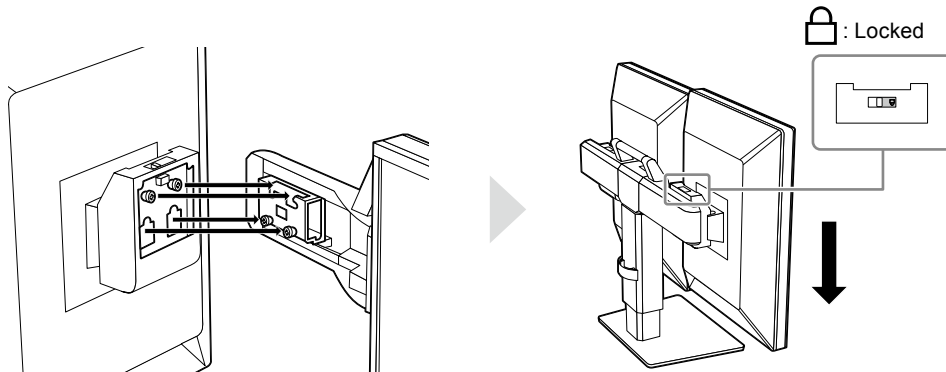Attention

- Check that the mount lock indicator is in the locked position.

Mechanical Specifications (RX560 and GX560)

Dimensions (W x H x D)	709.0 mm x 476.0 mm - 566.0 mm x 225.0 mm (Tilt: 0°, swivel (each monitor unit) : 0°) 696.5 mm x 476.0 mm - 566.0 mm x 237.1 mm (Tilt: 0°, swivel (each monitor unit) : 15°)
Net weight	RX560: 17.3 kg GX560: 17.1 kg
Height Adjustment	90 mm (Tilt: 0°)
Tilt	Up 25°, down 5°
Swivel (total)	40°
Swivel (each monitor unit)	15°

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