

# User's Manual FP-2101


Panel Protector for EIZO 54 cm (21.3) inch class LCD Monitors

## Attention

- Please be careful not to scratch the LCD panel or the cabinet with the panel protector during installation. Also please do not remove the panel protector often. It may scratch or result in damage to the LCD panel.
- The panel protector has a non-glare treatment side and a glossy side. Use preferable side.
- When using LCD monitor with optional panel protector, it is preferable to avoid using downwards. In this condition, the panel protector might become bend or slip out from the monitor.

## Installation

Before installation, please remove dust and stains from the LCD panel.

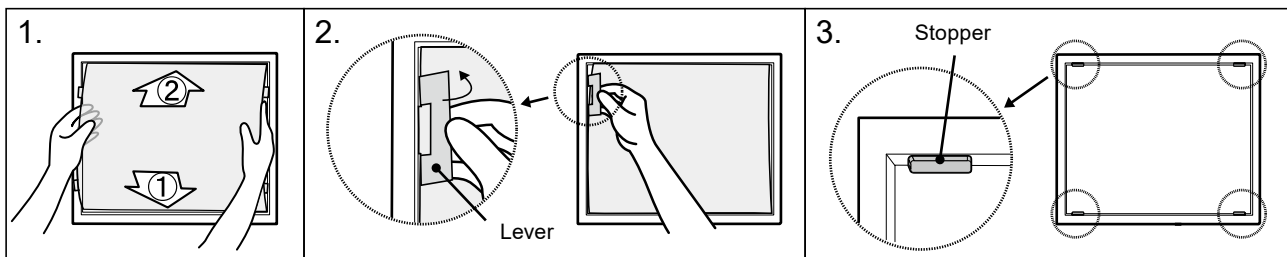

1. Set the panel protector into the grooves of the panel frame; the lower first (1), then the upper (2).
2. With the enclosed tool (Lever) set the tabs of the panel protector into the side grooves (4 tabs).  
Put the lever on the tabs of the panel protector and move it toward the LCD panel to install. Position the protector into the LCD panel tightly without leaving an opening.
3. Attach stoppers to the cabinet when installing a protection panel to the MX217.  
Peel the backing paper from the stoppers and install them in 2 locations in the upper part of the cabinet, and 2 locations in the lower part of the cabinet.

## Removing

Put the lever on the tabs of panel protector and move it toward the cabinet to remove.

## Cleaning

The panel surface can be cleaned with a soft cloth such as cotton or lens paper.

If necessary stubborn stains can be removed by moistening part of the cloth with water to enhance its cleaning power.

## Attention

- Please be sure to keep the panel protector immobilized while cleaning it. Otherwise it may scratch the surface of the LCD panel. To hold the panel protector by hand, use gloves or a soft cloth to avoid dust or finger prints.
- Chemicals such as alcohol and antiseptic solution may cause gloss variation, tarnishing, and fading of the cabinet or panel, and also quality deterioration of the image.
- Never use any solvents or chemicals, such as thinner, benzene, wax, and abrasive cleaner, which may damage the cabinet or panel.

## Note

- The optional ScreenCleaner is recommended for cleaning the cabinet and panel surface.

## Specification

Surface treatment	Non-glare side: non glare hard coating / Glossy side: hard coating
Coating strength	2H
Light transmission	More than 90 %
Dimension / Weight	435.3 mm (W) × 327.4 mm (H) × 1.0 mm (D) / Approx.175g
Accessories	Lever (for Panel Protector), User's Manual, Stoppers 4 pieces



## **EIZO Corporation**

153 Shimokashiwano, Hakusan, Ishikawa 924-8566 Japan

## **EIZO Europe GmbH**

Belgrader Straße 2, 41069 Mönchengladbach, Germany

## **EIZO 株式会社**

〒 924-8566 石川県白山市下柏野町 153 番地

[www.eizoglobal.com](http://www.eizoglobal.com)

[www.eizo.co.jp](http://www.eizo.co.jp)

Copyright © 2023 EIZO Corporation. All rights reserved.

00N0N352A1

UM-FP-2101

1st Edition - February, 2023 Printed in Japan.