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LS-HM1-D

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User's Manual LS-HM1-D

Dual Height Adjustable Stand


Before installation, visit our web site at <http://www.eizo.com> to confirm the applicable EIZO LCD monitor and the installation requirements of this stand.

Thank you very much for choosing the EIZO product.
Please read this User's Manual carefully to familiarize yourself with safe and effective usage.
Please retain this manual for future reference.

[Location of the Caution Statements (Bottom)]

PRECAUTIONS

To ensure personal safety and proper maintenance, please read this section and the caution statements on the unit.

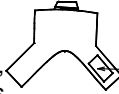WARNING: Do not disassemble. Injury may result.

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WARNING

Attach the applicable EIZO LCD monitors only.

Not doing so may cause the stand or monitor to drop, which may result in injury or equipment damage.

Attach the two monitors.

Attaching a monitor only may cause the stand to fall and result in injury or equipment damage.

Attach the LCD monitor securely to the stand.

Install the monitor securely with using the enclosed screws in the stand package. Follow the instructions on the user's manual. Not doing so may cause the monitor to come unattached, which may result in injury or equipment damage.

Do not disassemble.

The spring mechanism inside of the stand may result in injury or equipment damage when disassembling.


Retighten any screws that have loosened during use.


CAUTION

Carry the stand with holding the arm of the stand from underneath as shown in the below figures.

Not doing so may result in the equipment damage.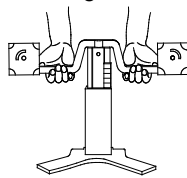

Before attaching

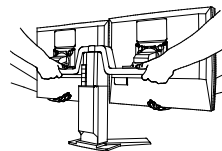Carry with another person after attaching.

Work with another person when attaching and adjusting height position.

Not doing so may result in injury or equipment damage.

Be careful not to pinch a finger when adjusting height and horizontal position.

When rotating the monitor for portrait display, make a space enough between a desk and a monitor not to pinch a finger.

Do not adjust the height or the angle with gripping the *i-Sound*, optional speaker (if applicable).

Adjusting the height or the angle with gripping the *i-Sound* is dangerous. It may result in injury or equipment damage.

Package Contents

Please contact your local dealer for assistance if any of the listed items are missing or damaged.

- Stand
- Stand Mounting Cover, 4 pcs
- Stoppers for height adjustment (attached to the stand, detachable), 5 pcs
- User's Manual
- Mounting Screws, 8 pcs each
Refer to "Support > Product Database > Accessories > LS-HM1-D > Manuals" on the EIZO website for compatible screws to use for installation.
<http://www.eizoglobal.com/support/db/products/model/LS-HM1-D>


Note before Use

The paperboard is inserted on the stoppers of this stand for the protection of the stand in transport condition. Remove the paperboard before use. The procedures are as follows.

1. Detach the bottom one in five stoppers.
2. Slide down other four stoppers and remove the paperboard.
3. Reattach the detached stopper.


NOTE

- When repacking and transporting the stand, reinsert the paperboard on the stoppers for the protection of the stand.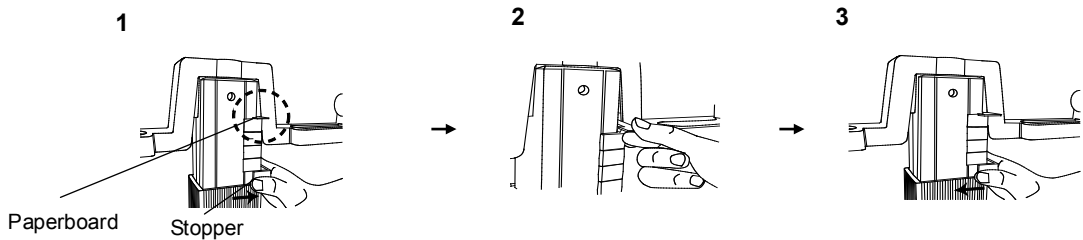

How to attach


1. Attach the covers to the shield plate with sliding them rightward or leftward.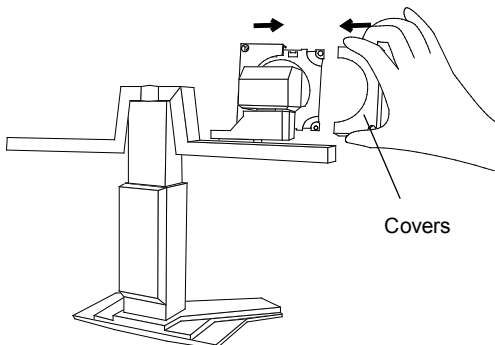

NOTE

- Use specified screws for tightening up referring to the our web site. <http://www.eizoglobal.com>


2. Position the monitor to fit the holes at the rear of the monitor to the shield plate of the stand. Tighten the two screws at the top as a temporary stop then tighten the rest of screws. (Tighten the two screws at the top securely after attaching all screws.)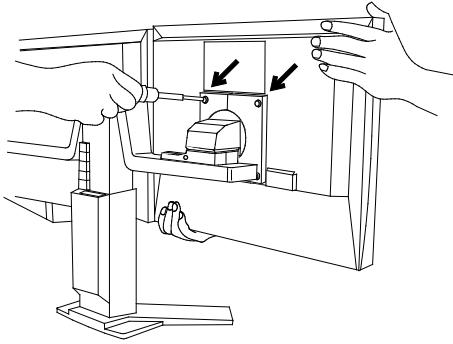

How to adjust


Height Adjustment

Adjust the height with using the stoppers.
 (Six stage height adjustable)
 Attach or remove the stopper from the bottom one by one with holding the arm of stand from underneath.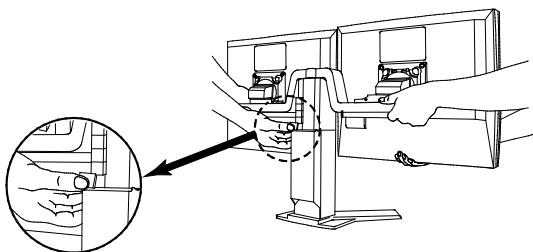


Horizontal Position Adjustment

Grip the top and bottom sides of the monitor firmly and slide it with lifting the bottom side of the monitor to the front.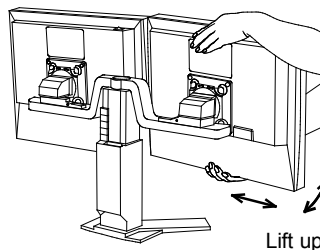


Angle Adjustment

Hold the monitor with both hands and adjust the monitor angle as desired.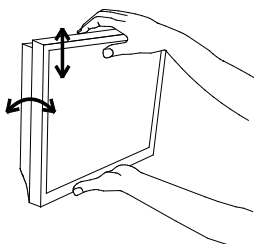

Cable Connection

Connect the cables (ex. signal cable) to the monitor.
 After connecting, lead the cables into the cable holder.
 (The cables are recommended to lead with slight sag for the smooth motion of the stand when the height, horizontal position and angle adjustments.)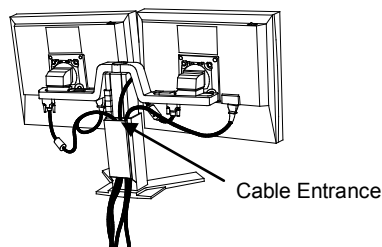


Specifications

Height Adjustment Range	75 mm [Six stage height adjustment: bottom point, +15, +30, +45, +60, +75 mm]
Dimensions	604 (W) x 326 to 401 (H) x 212 (D) mm
Weight	6.8 kg

Dimensions

Unit: mm

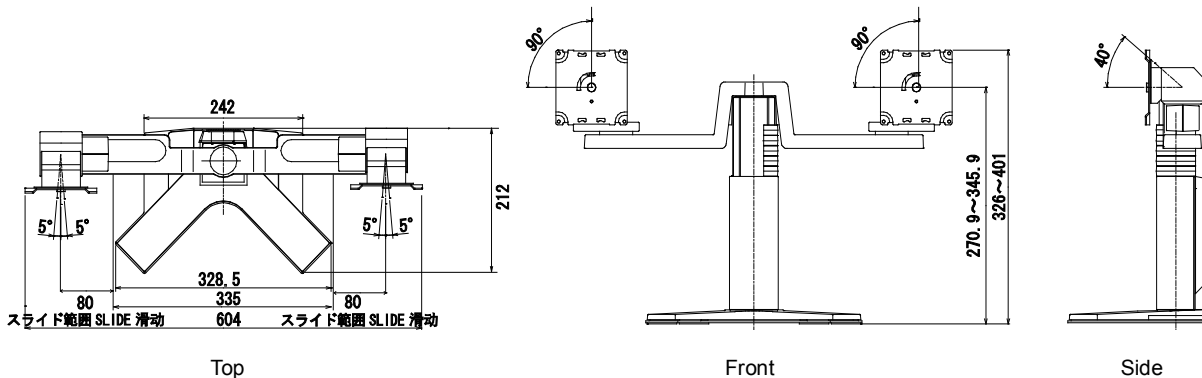LIMITED WARRANTY

EIZO Corporation (hereinafter referred to as “**EIZO**”) and distributors authorized by EIZO (hereinafter referred to as the “**Distributors**”) warrant, subject to and in accordance with the terms of this limited warranty (hereinafter referred to as the “**Warranty**”), to the original purchaser (hereinafter referred to as the “**Original Purchaser**”) who purchased the product specified in this document (hereinafter referred to as the “**Product**”) from EIZO or Distributors, that Product shall be free from defects in material and workmanship for a period of one (1) year from the date of purchase.

EIZO and Distributors will cease to hold or store any parts of the Product upon expiration of five (5) years after the production of such parts is discontinued. If the unit cannot be repaired due to its condition or the stockout of a relevant part, EIZO and Distributors may offer the replacement by a product with equivalent performance instead of repairing it.

This warranty is valid only in the countries or territories where the Distributors are located.

In the event of malfunction of the Product during the warranty period attributable directly to defective material and/or workmanship, EIZO and Distributors will, at their option, either repair or replace the defective product.

Notwithstanding any other provision of this Warranty, EIZO and Distributors shall have no obligation under this Warranty whatsoever in any of the cases as set forth below:

- (a) Any defect of the Product caused by freight damage, modification, alteration, abuse, misuse, accident, incorrect installation, disaster, faulty maintenance and/or improper repair by third party other than EIZO and Distributors;
- (b) Any incompatibility of the Product due to possible technical innovations and/or regulations;
- (c) Any deterioration of the sensor;
- (d) Any defect of the Product caused by external equipment;
- (e) Any defect of the Product on which the original serial number has been altered or removed;
- (f) Any normal deterioration of the Product, particularly that of consumables, accessories, and/or attachments (e.g. buttons, rotating parts, cables, User’s Manual, etc.); and
- (g) Any deformation, discoloration, and/or warp of the exterior of the Product including that of the surface of the LCD panel.

To obtain service under the Warranty, the Original Purchaser must deliver the Product, freight prepaid, in its original package or other adequate package affording an equal degree of protection, assuming the risk of damage and/or loss in transit, to the local Distributor. The Original Purchaser must present proof of purchase of the Product and the date of such purchase when requesting services under the Warranty.

The Warranty Period for any replaced and/or repaired product under this Warranty shall expire at the end of the original Warranty Period.

EIZO OR DISTRIBUTORS ARE NOT RESPONSIBLE FOR ANY DAMAGE TO, OR LOSS OF, DATA OR OTHER INFORMATION STORED IN ANY MEDIA OR ANY PART OF ANY PRODUCT RETURNED TO EIZO OR EIZO’S AUTHORIZED DISTRIBUTORS FOR REPAIR.

EIZO AND DISTRIBUTORS MAKE NO FURTHER WARRANTIES, EXPRESSED OR IMPLIED, WITH RESPECT TO THE PRODUCT AND ITS QUALITY, PERFORMANCE, MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR USE. IN NO EVENT SHALL EIZO OR DISTRIBUTORS BE LIABLE FOR ANY INCIDENTAL, INDIRECT, SPECIAL, CONSEQUENTIAL OR OTHER DAMAGE WHATSOEVER (INCLUDING, WITHOUT LIMITATION, DAMAGES FOR LOSS OF PROFIT, BUSINESS INTERRUPTION, LOSS OF BUSINESS INFORMATION, OR ANY OTHER PECUNIARY LOSS) ARISING OUT OF THE USE OR INABILITY TO USE THE PRODUCT OR IN ANY CONNECTION WITH THE PRODUCTS, WHETHER BASED ON CONTRACT, TORT, NEGLIGENCE, STRICT LIABILITY OR OTHERWISE, EVEN IF EIZO OR DISTRIBUTORS HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. THIS EXCLUSION ALSO INCLUDES ANY LIABILITY WHICH MAY ARISE OUT OF THIRD PARTY CLAIMS AGAINST THE ORIGINAL PURCHASER. THE ESSENCE OF THE PROVISION IS TO LIMIT THE POTENTIAL LIABILITY OF EIZO AND DISTRIBUTORS ARISING OUT OF THIS LIMITED WARRANTY AND/OR SALES.

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Copyright© 2003-2015 EIZO Corporation All rights reserved.
EIZO GmbH