# RadiForce® MS170

Color LCD Monitor

English

Deutsch

Français

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## Setup Manual

**Important:** Please read this Setup Manual and the User's Manual stored on the CD-ROM carefully to familiarize yourself with safe and effective usage. Please retain this manual for future reference.

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Color LCD Monitor

## SAFETY SYMBOLS

This manual uses the safety symbols below. They denote critical information. Please read them carefully.

 <b>WARNING</b> Failure to abide by the information in a WARNING may result in serious injury and can be life threatening.	 <b>CAUTION</b> Failure to abide by the information in a CAUTION may result in moderate injury and/or property or product damage.
 Indicates a prohibited action.	
 Indicates a mandatory action that must be followed.	

## PRECAUTIONS

### WARNING

**If the unit begins to emit smoke, smells like something is burning, or makes strange noises, disconnect all power connections immediately and contact your dealer for advice.**

Attempting to use a malfunctioning unit may result in fire, electric shock, or equipment damage.

**Use the enclosed power cord and connect to the standard power outlet of your country.**

Be sure to remain within the rated voltage of the power cord. Not doing so may result in fire or electric shock.

"Power Supply: 100-120/200-240 Vac 50/60Hz"

**Keep small objects or liquids away from the unit.**

Small objects accidentally falling through the ventilation slots into the cabinet or spillage into the cabinet may result in fire, electric shock, or equipment damage. If an object or liquid falls/spills into the cabinet, unplug the unit immediately. Have the unit checked by a qualified service engineer before using it again.

**Use the unit in an appropriate location.**

Not doing so may result in fire, electric shock, or equipment damage.

- Do not place outdoors.
- Do not place in the transportation system (ship, aircraft, trains, automobiles, etc.).
- Do not place in a dusty or humid environment.
- Do not place in a location where water is splashed on the screen (bathroom, kitchen, etc.)
- Do not place in a location where the steam comes directly on the screen.
- Do not place near heat generating devices or a humidifier.
- Do not place in an inflammable gas environment.

**The equipment must be connected to a grounded main outlet.**

Not doing so may result in fire or electric shock.

**Fluorescent backlight lamps contain mercury (the products that have LED backlight lamps contain no mercury), dispose according to local, state or federal laws.**

### CAUTION

**Do not block the ventilation slots on the cabinet.**

- Do not place any objects on the ventilation slots.
  - Do not install the unit in a closed space.
  - Do not use the unit laid down or upside down.
- Blocking the ventilation slots prevents proper airflow and may result in fire, electric shock, or equipment damage.

**Use an easily accessible power outlet.**

This will ensure that you can disconnect the power quickly in case of a problem.


## Package Contents

Check that all the following items are included in the packaging box. If any items are missing or damaged, contact your local dealer.


### NOTE

Please keep the packaging box and materials for future movement or transport of the monitor.


- Monitor
- Power cord
- Digital signal cable : FD-C39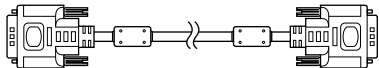

- Analog signal cable : MD-C87

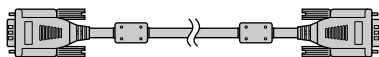
- EIZO USB cable : MD-C93

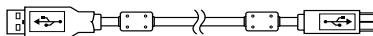- EIZO LCD Utility Disk (CD-ROM)
  - RadiCS LE (for Windows)
  - ScreenManager Pro for Medical (for Windows)
  - User's Manual
- Setup Manual (this manual)
- Limited warranty
- Recycling Information

## Controls and Functions

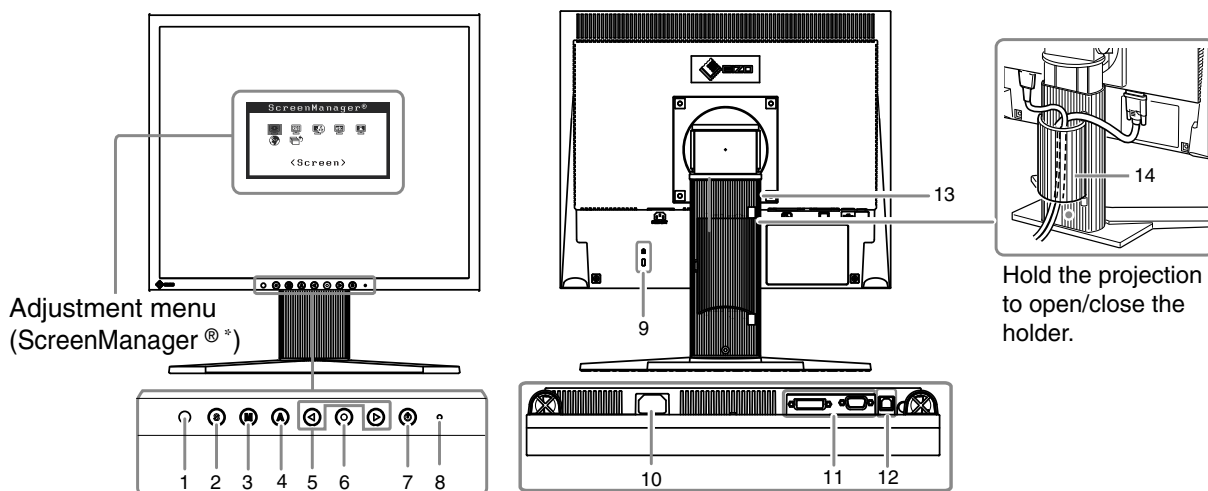1	<b>Sensor</b>	Detects ambient brightness. Auto EcoView function. Refer to the User's Manual on the CD-ROM.
2	<b>Input Signal Selection button</b>	Switches input signals for display when two PCs are connected to the monitor.
3	<b>Mode button</b>	Displays the CAL Switch menu (page 5).
4	<b>Auto button</b>	Performs the function to adjust the screen automatically. (analog input only)
5	<b>Control buttons (Left, Right)</b>	Chooses an adjustment item or increases/decreases adjusted values for advanced adjustments using the Adjustment menu.
6	<b>Enter button</b>	Displays the Adjustment menu, determines an item on the menu screen, and saves values adjusted.
7	<b>Power button</b>	Turns the power on or off.
8	<b>Power indicator</b>	Indicates monitor's operation status. Blue: Operating Orange: Power saving Off: Power off
9	<b>Security lock slot</b>	Complies with Kensington's MicroSaver security system.
10	<b>Power connector</b>	Connects the power cord.
11	<b>Input signal connectors</b>	Left: DVI-D connector / Right: D-sub mini 15-pin connector.
12	<b>USB port (Up)</b>	Connects the USB cable to use the software that needs USB connection.
13	<b>Stand</b>	Used to adjust the height and angle of the monitor screen.
14	<b>Cable holder</b>	Covers the monitor cables.

\* ScreenManager<sup>®</sup> is an EIZO's nickname of the Adjustment menu. (For how to use ScreenManager, refer to the User's Manual on the CD-ROM.)


# Connecting Cables

## Note

- Check that the monitor and the PC are powered off.
- When replacing the current monitor with a MS170 monitor, be sure to change the PC settings for resolution and vertical frequency to those which are available for the MS170 monitor referring to the resolution table (back of cover page) before connecting the PC.

## Tips

- When connecting two PCs to the monitor, refer to the User's Manual on the CD-ROM.

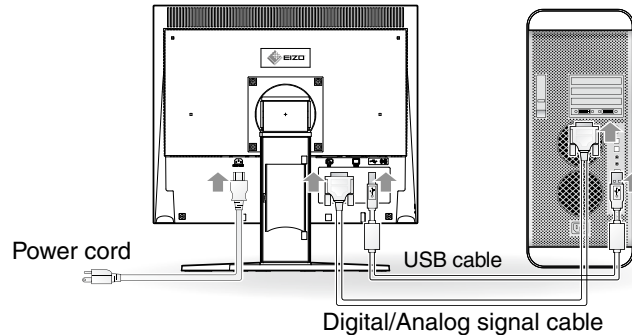- 1 Connect the monitor to the PC with a signal cable that matches the connectors.**  
After connecting the cable connectors, tighten the screws of the connectors to secure the coupling.

### Digital Input

Connectors on the monitor	cable	Connectors on the PC
	Signal cable FD-C39 	

### Analog Input

Connectors on the monitor	cable	Connectors on the PC
	Signal cable MD-C87 	

- 2 Connect the USB cable when using RadiCS LE or ScreenManager Pro for Medical.**

- 3 Plug the power cord into a power outlet and the power connector on the monitor.**

- 4 Press to turn on the monitor.**

The monitor's power indicator lights up blue.

- 5 Turn on the PC.**

The screen image appears.

When turning on the monitor and PC for the first time with the analog signal, the AutoAdjustment function works to adjust the clock, phase, and display position automatically.

If an image does not appear, refer to "No-Picture Problem" (page.6) for additional advice.


---

**Note**

- Turn off the monitor and PC after using them.
  - For the maximum power saving, it is recommended that the Power button be turned off. Unplugging the power cord completely shuts off power supply to the monitor.
- 

## Adjusting the Screen Height and Angle

Hold left and right edge of the monitor with both hands, and adjust the screen height, tilt and swivel of the screen to the best condition for working.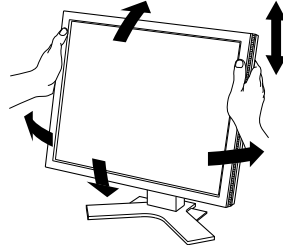

---

**Note**

- Be sure that the cables are correctly connected.
- 

## Installing RadiCS LE

The quality control software "RadiCS LE" (for Windows) is included with this monitor. To use RadiCS LE, install it with reference to the RadiCS LE User's Manual on the CD-ROM.

---

**Note**

- Connect the monitor to the PC with a USB cable when using RadiCS LE.
- 

## Adjusting Brightness

**1** Press ◀ or ▶.

The brightness adjustment screen appears.

**2** Adjust the brightness with ◀ or ▶.

Press ▶ to increase screen brightness or press ◀ to decrease.

---

**Tips**

- Screen will be adjusted automatically to the appropriate brightness that accepted environment as the sensor on the lower side of the monitor detects the surrounding brightness with the Auto EcoView function. For details, refer to the User's Manual on the EIZO LCD Utility Disk (CD-ROM).
-

# Selecting Display Mode

CAL Switch function allows you to select the best display mode easily according to monitor's application.

For details of CAL Switch function, refer to the User's Manual on the CD-ROM.

## CAL Switch Mode

<b>Custom</b>	Available for making desired setting.
<b>sRGB</b>	Suitable for color matching with sRGB compatible peripherals.
<b>Text</b>	Suitable for displaying texts for word processing or spreadsheets.
<b>DICOM</b>	Used to display images in the DICOM mode

### 1 Press .


The CAL Switch menu appears.

### 2 Press again while the CAL Switch menu is displayed.

Each time the button is pressed, the mode on the screen switches. (Refer to the CAL Switch Mode table.)

### 3 Press at the desired mode.

The selected mode is set.

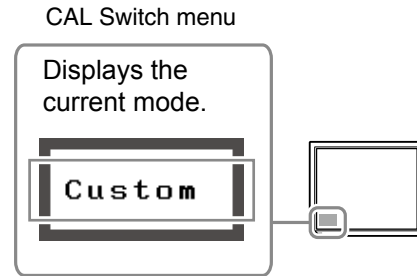The basic settings and adjustments are completed. For advanced settings/adjustments, refer to the User's Manual on the CD-ROM.

# No-Picture Problem


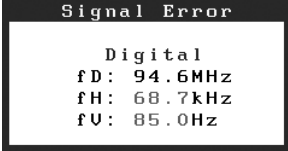
If no picture is displayed on the monitor even after the following remedial action is taken, contact your local dealer.

## 1. Check the Power indicator.

Symptom	Status	Possible cause and remedy
No picture	Power indicator does not light up.	<ul style="list-style-type: none"> <li>• Check whether the power cord is connected correctly. If the problem persists, turn off the monitor, and then turn it on again a few minutes later.</li> <li>• Press .</li> </ul>
	Power indicator lights blue.	<ul style="list-style-type: none"> <li>• Set each adjusting value in [Brightness], [Contrast] and [Gain] to higher level.</li> </ul>
	Power indicator lights orange.	<ul style="list-style-type: none"> <li>• Switch the input signal with .</li> <li>• Press a key on the keyboard or click the mouse.</li> <li>• Turn on the PC.</li> </ul>

## 2. Check the error message that remains on the screen.

These messages appear when the input signal is incorrect even if the monitor is functioning.

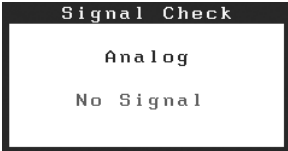Symptom	Status	Possible cause and remedy
	The input signal is not received.	<ul style="list-style-type: none"> <li>• Check whether the PC is turned on.</li> <li>• Check whether the signal cable is properly connected to the PC.</li> <li>• Switch the input signal with .</li> </ul>
	The signal frequency is outside the specification. Incorrect signal frequency is shown in red.	<ul style="list-style-type: none"> <li>• Reboot the PC.</li> <li>• Select an appropriate display mode using the graphics board's utility. Refer to the manual of the graphics board for details.</li> </ul> <p>fD : Dot Clock (Displayed only when the digital signal inputs) fH : Horizontal Frequency fV : Vertical Frequency</p>

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
## [Location of the Caution Statements]

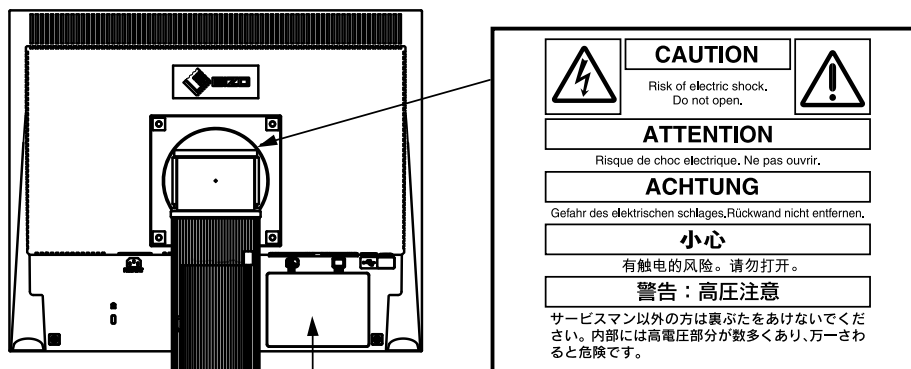The equipment must be connected to a grounded main outlet.  
Jordet stikkontakt skal benyttes når apparatet tilkobles datanett.  
Apparaten skall anslutas till jordat nätuttag.  
電源コードのアースは必ず接地してください。  
这设备必须连接至接地主插座。

---

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation.

EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

---

Apple and Macintosh are registered trademarks of Apple Inc.

VGA is a registered trademark of International Business Machines Corporation.

Windows is a registered trademark of Microsoft Corporation in the United States and other countries.

VESA is a registered trademark or a trademark of Video Electronics Standards Association in the United States and other countries.

PC-9801 and PC-9821 are trademarks of NEC Corporation.

EIZO, RadiCS, RadiForce and ScreenManager are registered trademarks of EIZO Corporation in Japan and other countries.

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<http://www.eizo.com>

Copyright © 2009-2013 EIZO Corporation All rights reserved.

2nd Edition-April, 2013 Printed in Japan.

00NOL540B1  
(U.M-SUMS170-4)

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## **EIZO Europe GmbH**